



# Wired Guardian MK3 Monitor Handbook

Guardian Type S1020A/1017A

For use with all Nurse Call Systems

---

[www.alert-it.co.uk](http://www.alert-it.co.uk)

---

## Introduction

The Alert-iT Guardian is a highly versatile monitor unit that employs a range of sensors to detect various presentations of seizure activity, before passing an alarm signal to a Nurse Call system. Your Guardian is preset to detect the key symptoms most commonly associated with tonic/clonic seizures, and has been supplied with sensors appropriate to your needs.

Before installing the Guardian, you will need to have sufficient knowledge of the client to make the necessary risk assessment as to the suitability of the equipment and sensors required to provide a safe environment with comprehensive support.

This handbook will help you install, test and adjust the settings of your Guardian and its sensors.

Once you have installed your Guardian, we will be pleased to offer you any assistance you require via our telephone help-line on 01530 239 900. Our working hours are 9am-5pm Monday to Friday.

Please test the operation of your Guardian and sensors at least once per week. You will find a simple form on page 29 which will help you keep track of this process.

Should you wish to change any of the operating parameters or modify your system, detailed handbooks are available on request (for hard copies), or on-line at: **[www.alert-it.co.uk/support](http://www.alert-it.co.uk/support)**

Please call 01530 239 900 or go online to **[www.alert-it.co.uk/registration](http://www.alert-it.co.uk/registration)** and register your new equipment with our Service Team. You will be assigned an SRN (Support Registration Number) which will link your details and equipment to our database, ensuring that you always receive fast and efficient service and support. An additional benefit is that this will extend your warranty to three years at no extra charge.

## Contents

Product Information	5
Client Assessment	5
The Guardian and its Optional Sensors	6
Using The Guardian	7
Part 1 - Quick-Start Guide	7
Switching The Unit On	7
Sensor Installation and Basic Operation	8
Output to Nurse Call Systems	8
Spasm Movement	9
<i>Install the Sensor</i>	9
<i>Activate the Function</i>	9
<i>Test the sensor</i>	9
<i>Normal Operation</i>	9
Shallow Movement	10
<i>Install the Sensor</i>	10
<i>Activate the Function</i>	10
<i>Test the Sensor</i>	11
<i>Normal Operation</i>	11
Sound	12
<i>Install the Sensor</i>	12
<i>Activate the Function</i>	12
<i>Test the Sensor</i>	12
<i>Normal Operation</i>	12
Bed Occupancy (if applicable)	13
<i>Install the Sensor</i>	13
<i>Activate the Function</i>	13
<i>Test the Sensor</i>	13
<i>Normal Operation</i>	13
Moisture (if applicable)	14
<i>Install the Sensor</i>	14
<i>Activate the Function</i>	14
<i>Test the sensor</i>	14
<i>Normal Operation</i>	14
Alarm Condition	15
Suspend Mode Explained	16

Cont'd Overleaf

Part II - User Settings & Adjustments	17
Spasm	17
Enabling Shallow Delay	18
Enabling Shallow Minimum	18
Shallow Magnify	18
Tick Volume (Audible Breathing Sound)	18
Recommended Procedure For Setting Shallow Movement Alarms	19
Shallow Movement Adjustment Procedure	20
Bed Vacation	22
Sound Settings	23
<i>Sound Delay</i>	23
<i>Sound Magnify</i>	23
<i>Sound Rate</i>	23
Moisture	24
Other Settings	25
<i>Snooze Function Zz</i>	25
<i>Reload Default Settings</i>	25
<i>Auto-test of sensors</i>	25
Maintenance	27
Compliance	27
Bibliography	27
<i>Appendix A - Remote Reset Mode</i>	27
Important Safety Information	28
Routine Testing Sheet	29



## Product Information

The Alert-iT Guardian is without doubt the most versatile and sensitive epilepsy support monitoring system ever developed for care home use.

To get the most from your monitor, we recommend that on first installing it, you use only the tonic/clonic seizure functions for the first night. Setting the monitor up for this key role is an ideal introduction to using the controls and understanding the information on the display screen. You will then be able to progress quickly to using the other functions as described later in this handbook.

In addition, our telephone Customer Support Team are here to assist you and can be contacted on 01530 239900.

The Guardian's wide range of settings, adjustments and nightly movement readings support the following symptoms:

**Tonic/clonic seizures from tremors to large movements**

**SUDEP through failure of shallow movements associated with breathing**

**Breathing problems through changes in shallow movement activity**

**Enuresis, vomiting and salivation via the moisture sensor sheets**

**Immediate or prolonged bed vacation**

**Sharp vocal sounds associated with seizures**

The appropriate sensors for these symptoms can be verified on the table featured below.

### Client Assessment


Symptoms					
Sensor	Bed Movement	Enuresis (moisture, vomit or salivation)	Bed Vacation	Shallow Movement (Associated with breathing)	Sound
P140 Bed Movement Sensor	✓			✓	
P143C Bed Occupancy Mat (S1016 only)			✓		
P142A/F Moisture Sensors		✓			
Internal Guardian Microphone or P158B					✓

## The Guardian and its Optional Sensors

A	Bedside Monitor (Guardian)	P139*
B	Bed Movement Sensor (Included)	P140A
C	Connecting Lead for F (Included with S1017)	P141E(S1017)
D	Bracket for wall or bed mounting (Included)	P159A
E	Power Supply for Guardian (Included)	P171*

### **Additional Items:**


F	Moisture Sheet(s) (optional purchase)	P142A/F
G	Nurse Call Adaptor Lead (purchased separately)	P145**
H	Bed Occupancy Sensor (Optional Extra)	P143*
I	External microphone (optional purchase)	P158B

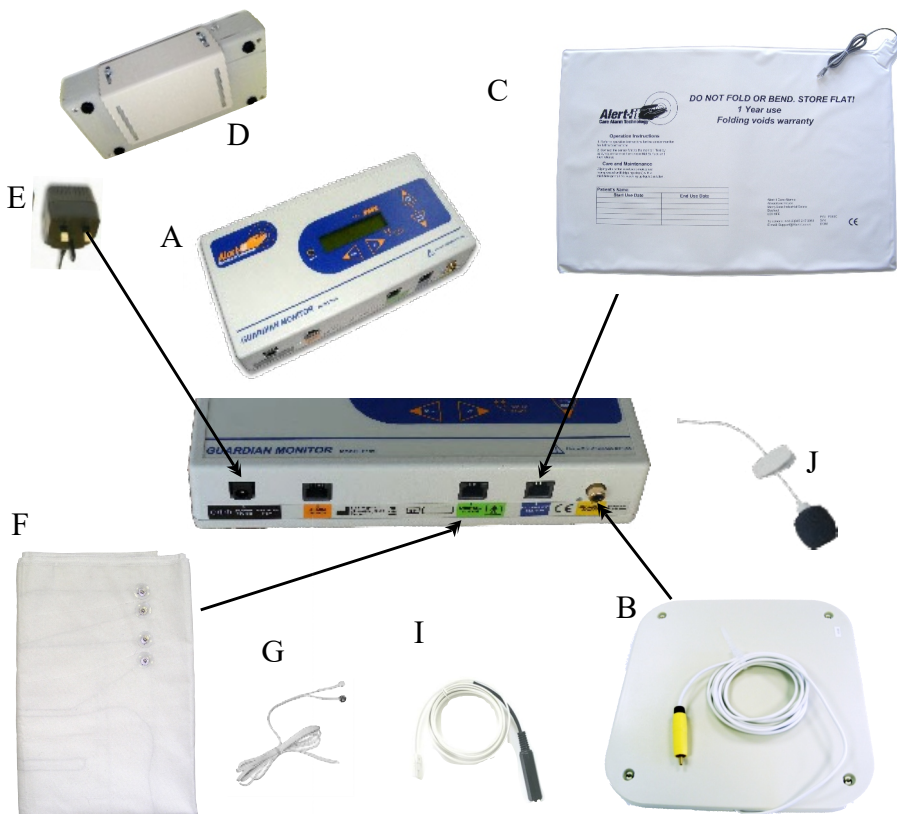


# Part 1 - Quick-Start Guide

## Using The Guardian

### Basic controls: Standby

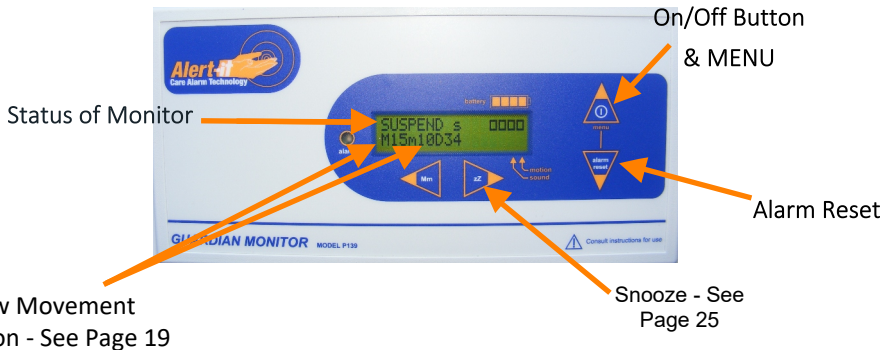

### Basic controls: Adjustment Example

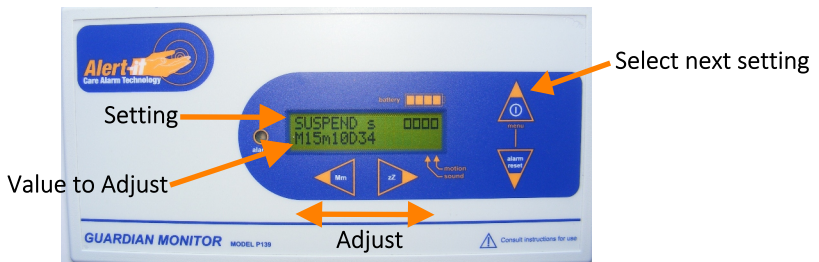

## Switching The Unit On

Insert the power supply into the socket on the Guardian marked 'POWER 12V DC'. (Fig.1). Press the arrow button marked 'menu' to switch the Guardian on.

The Guardian incorporates an internal battery, ensuring that the monitor has a constant power source in case of power cuts. This battery will charge whenever the power supply is plugged in. If the battery indicator rectangles appear as solid shapes, the unit is charging; if they appear as outlines only, the unit is operating on its battery power alone.

If the battery becomes discharged then the monitor will emit an alarm noise (which sounds like a croaking frog) and a fault message will be transmitted to your Nurse Call System or Third Party Telecare System.




Power Supply Socket

Figure 1

## Sensor Installation and Basic Operation

The various sensors are positioned in and around the bed as shown below. The following sections of this handbook are presented in the recommended sequence of first use of each sensor, introducing each new sensor in turn, and only activating each new sensor after you have one or two night's use of the previous one. This methodical approach is highly encouraged, as it gives you time to familiarise yourself with the way each sensor operates, and ensures that you will go on to use your Guardian confidently, safely and to its full potential.

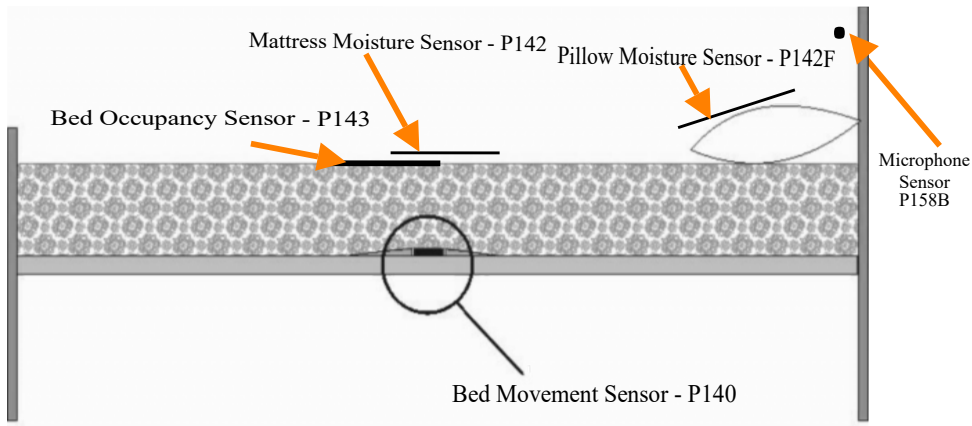

Positioning sensors on the bed Fig 2

## Output to Nurse Call Systems

The Guardian is fitted with a six-pin multi-function Nurse Call socket which is compatible with all systems. A matching lead for your particular system (P145\*\*) will need to be ordered with your Guardian. Connect this lead via the orange 'Alarm Output' socket.



Figure 3

## Spasm Movement

### Designed to detect tonic/clonic seizures



Figure 4

#### ***Install the Sensor***

Place the Bed Movement Sensor (P140A) underneath the mattress on a firm bed base, in a position below the torso (as per fig. 2). Connect the sensor into the Guardian monitor making sure that the sensor's yellow plug connects to the movement sensor plug (also marked with yellow as per figure 4).

#### ***Activate the Function***

The Guardian is factory-set with this feature activated. It is pre-set for sending an alarm after detecting tonic/clonic movements lasting for longer than 15 seconds, at a rate of at least one movement every two seconds. To deactivate or alter these settings, please see page 17.

#### ***Test the sensor***

Tap the mattress continuously and note that the # symbol appears and remains visible as you tap for the duration of the delay time - this is 15 seconds at the factory setting.

#### ***Normal Operation***

An alarm will occur if continuous movements exceed the rate and time settings and a signal will be passed to your Nurse Call System or Third Party Telecare System. It is important to ensure that the Guardian resumes its ACTIVE mode once the alarm has been reset and the occupant has returned to the bed.

## Shallow Movement

### Designed to support breathing-related seizures and SUDEP

#### *Install the Sensor*

Shallow movement is detected using the P140 Bed Movement sensor, which will have already been installed (see page 9).

#### *Activate the Function*

By default this is OFF on delivery. Use the MENU key to scroll to “Shallow Minimum” and activate the function by pressing the Zz arrow. There are then four parameters to set:

- **Shallow Magnify (Sensitivity):** The higher the number the smaller the movements detected, which reduces false alarms. However, the sensor **MUST** be tested for immunity to ambient vibrations (see page 19-21)
- **Shallow Delay (Identified as d on the display):** How long the client stops moving totally to generate an alarm. At the start this is set to 60 seconds and refined later with observation of the clients general breathing pattern (see page 19-21)
- **Shallow Minimum (Identified as m on the display):** The trigger level for slow breathing. At the start this should be set to 0 and refined later with observation of the clients general breathing pattern (see pages 19-21)
- **Shallow Maximum (Identified as M on the display):** The trigger level for fast breathing. At the start this should be set to off and refined later with observation of the clients general breathing pattern (see pages 19-21)

When setting these parameters to give fast reliable detection with minimum false alarms, the client's mattress type, physique and nighttime breathing patterns all need to be taken into account and adjusted for. To help you achieve this, we strongly recommend using the procedure on page 19 which uses the built-in movement recorder overnight to help establish the monitor's optimum settings.

### ***Test the Sensor***

A useful test facility is the “Tick Volume”. This is the second option from the home screen as you scroll through the main menu. It is set to OFF by default, but after changing to 8 (full volume) a tick will be heard at every detected breath or shallow movement. To test the sensor; lie on the bed and note that a tick is heard and the breathing symbol (\*) appears on the display at every breath.

### ***Normal Operation***

If any of the selected alarm conditions are exceeded, an alarm will be raised. After this alarm is cleared, further alarms are suspended until a client is once again detected in bed (see explanation page 16). This prevents false alarms while the bed is unoccupied.

In-bed (active) detection occurs if:

- The sensor detects 10 shallow movements within one minute
- A Bed Occupancy mat is activated (see page 13)

## **Safety Note**

---


The alarm is inhibited if the Bed Occupancy mat indicates the client has left the bed. This could stop the alarm activating if, for instance, the client curls up at the bottom of the bed away from the mat or falls from the bed prior to a seizure. If this is possible then the Bed Occupancy mat should either not be used or only set to alarm after only a few seconds to reduce any long-term risk.

## Sound

### Designed to detect seizures associated with vocal sounds

#### *Install the Sensor*

The Guardian features an internal microphone. However, an external microphone is also available and may be required to focus sound detection if the monitor is being used in a noisy environment (part number P158B - sold separately).

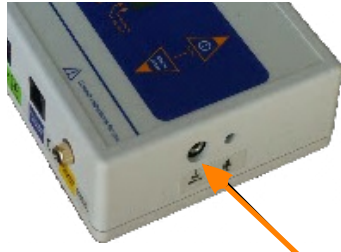

External Microphone Input

#### *Activate the Function*

This is turned off at factory settings. To activate the sound detection function, use the MENU key to scroll to "SOUND". There are then three parameters to set:

- **Sound Delay:** How long the sound sequence must go on for to trigger an alarm.
- **Sound Magnify:** This ranges from 1-8. The higher the number selected, the quieter the sound detected.
- **Sound Rate:** How frequent the shouting or clicking has to be per minute.

#### *Test the Sensor*

Make a sharp click near the sensor and check that a ) symbol appears on the display. If no ) appears, the sensitivity (Sound Magnify 1-8) should be increased until the sound registers and the symbol appears.

#### *Normal Operation*

If the sound pattern matches the rate and carries on longer than the set delay, an alarm is generated. The alarm is cancelled and cleared using your Nurse Call System or Third Party Telecare System.

\*If the client is detected out of bed (with either the Bed Occupancy Mat - see page 13, or the Shallow Movement detector enabled - see page 10), then the microphone will be inhibited to prevent false alarms.



## Bed Occupancy (if applicable)

### Install the Sensor

Insert the sensor cable into the Guardian's Occupancy Monitor socket as shown in figure 5. Place the Bed Occupancy Sensor (P143C) on top of the mattress under a suitable cover sheet, in a position that ensures the maximum body weight is lying on the sensor. Typically this is the area beneath the upper torso. Positioning the sensor underneath the shoulder area is advised if an alarm is required before the user's feet touch the floor.



Figure 5

### Activate the Function

The Guardian has this function turned off as its factory setting. To turn the function on, use the MENU key to scroll to "Bed Occupancy" and use the right-hand arrow (Zz) to select the time required for the occupant to be missing before the alarm is raised. A typical setting\* would be 6-10 minutes to allow for a toilet visit, yet avoiding the danger of leaving the client unattended should a seizure occur in the bathroom (however, please see the safety comment regarding SUDEP on page 11). Please be aware - this sensor has safety implications when used in conjunction with the Shallow Movement or Sound functions (see pages 10 & 12)\*

### Test the Sensor

With the mat plugged in and the bed unoccupied, the word 'vacant' should appear on the display. Lie on the bed and 'in-bed' should replace the 'vacant' message.

### Normal Operation

When the occupant leaves the bed, this sensor will automatically inhibit the shallow movement and sound alarms. If the occupant fails to return to the bed within the prescribed time the alarm is raised. The alarm resets automatically if the client returns. You may also cancel an alarm manually via your Nurse Call System or Third Party Telecare System.

\*Please note. Any settings must be supported by the risk assessment carried out for the individual.

## Moisture (if applicable)

### ***Install the Sensor***

Connect the Moisture Sensor (P142) to its connecting lead using the press-studs (you must connect the studs to either the right or left-hand pair in the case of the mattress sheet). Next, plug the lead into the Guardian's green port (as shown in figure 6).



Figure 6



Please note that there are two types of sensor. They are either fitted over the pillow for vomit detection (P142F), or on top of the mattress (on top of the Bed Occupancy mat if fitted) in the region of the groin for enuresis monitoring (P142A).

### ***Activate the Function***

The Guardian has moisture detection turned off as a factory setting. To turn the function on, press the MENU key repeatedly until the "MOISTURE" option appears, and then use the right key (Zz) to adjust the degree of wetness required before the alarm is raised.

### ***Test the sensor***

To test the sensor, join the two spare stud connections (either left or right-hand pair) with a metal object to trigger an alarm.

### ***Normal Operation***

When the sensor sheet becomes wet, the alarm will sound. Press 'Reset' on the Guardian to silence and suspend (see explanation page [16](#)) the alarm until a dry moisture sensing sheet has been connected.

## Alarm Condition

Alarms are passed to a Nurse Call or Telecare system via a two second pulse.

Spasm Movement, Shallow Movement and Bed Vacation Alarms all feature a count function which indicates how long the alarm has been sounding. This information is displayed on the Guardian's LCD screen

Once the user has been attended to and the alarm cancelled on your system, it is strongly recommended that you check the Guardian has returned to "ACTIVE" before leaving the user, as this ensures an alarm condition will be detected.



Type of alarm currently active

Duration of alarm currently active  
(in seconds)

## Suspend Mode Explained

The word “active” on the main screen confirms that all the sensors are working and have sensed a safe condition for the bed occupant.

When the Alarm Reset Key is pressed following an alarm, or indeed at any time, sensors that could give false alarms when the occupant is out of the bed are temporarily suspended. They will then remain suspended until the occupant is sensed as being in the bed once again whereupon they will return to active mode automatically. This applies to the Shallow Movement, Sound and Bed Occupancy sensors. In addition the Moisture sensor is suspended until a dry sensor is detected.

If any sensor is suspended the the screen shows “Suspend” followed by the corresponding letter(s) indicating which sensors are suspended. All others sensors will still be active.


The following table defines the conditions that cause the sensor to become active.

	Sensor	Action to trigger Active mode	Effect when Suspended
s	Shallow	After 10 breaths or if Bed Occupied sensor active	Shallow alarms are not sensed when S is showing
s	Sound	After 10 breaths or if Bed Occupied sensor active (or always if one of these is not active)	Sounds are not sensed while S is showing
m	Moisture	When sensor dry	Moisture is not sensed while m is showing
b	Occupancy	When “in bed” sensed	The Bed Vacation alarm timer is stopped while b is showing.

## Part II - User Settings & Adjustments

### Spasm

Press the menu to scroll to the relevant spasm setting. Using the left and right arrow to alter settings sensitivity.

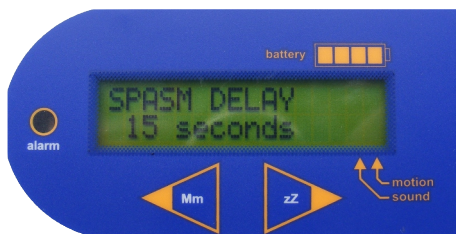

Spasm Delay is the amount of time the person is moving constantly before the alarm is triggered. The unit is supplied with a default setting of 15 seconds.

Spasm Magnify is the sensitivity adjustment for intensity of movement. 8 is most sensitive. Slight tremors will be setting 8. The unit is supplied with a default setting of 4.



Spasm Rate is set at a default of 2, meaning that the unit will alarm if there is 1 movement every 2 seconds (i.e. within the setting). Lowering spasm rate to 4 detects slower regular movements. E.g. Setting 4 could support, myclonic detection (1 movement every 4 seconds or faster, which triggers an alarm).



Shallow Maximum (default is Off) Starts at 20 MPM (movement/breaths per minute) maximum is 30. This is to detect rapid movements/breaths which may indicate panic attack or hyperventilation.



## Tick Volume (Audible Breathing Sound)

The tick volume can be adjusted from OFF to the maximum of 8. (Default=OFF). A soft yet distinct sound Tick volume enables carers to maintain a discreet audible vigilance of the user's breathing pattern.



## Enabling Shallow Minimum

Shallow Minimum monitors the average number of breaths taken per minute. When the function is turned ON and the person is in bed the Guardian should go into Active Mode within approximately 10 movements/breaths. It is important to ENSURE Active is displayed (and not suspend: see page 16) or the person will not be monitored for shallow movement and sound.



## Enabling Shallow Delay

Shallow Delay is activated when Shallow Minimum is ON. Shallow Delay will alarm at when breathing ceases after a set time. Adjustable but factory set at 30 seconds, it will therefore alarm after no movements/breaths are detected within any given 30 second period (note - this is independent of the the Shallow Minimum setting, and essentially functions acts as a failsafe feature to detect an absence of breathing movement.)



## Shallow Magnify


Shallow magnify is to allow for variants in the size and weight of a person, compared to mattress type & density. The highest setting will reduce false alarms, provided there are no extraneous vibrations being picked up when no-one is in bed (such as from an airflow mattress).







## Recommended Procedure For Setting Shallow Movement Alarms


1. Using the TICK function is a good way to audibly check that the movement sensor is working. Turn the function on and select setting 8.
2. Set the Shallow Min to 0, Shallow Magnification at 8, Shallow Delay=15 and with no user in the bed and no one touching the bed, check that neither the \* or # symbol do NOT appear on the screen. Then leave the unit in SUSPEND (press RESET key) for an hour with no-one in the room and check it remains in SUSPEND. Decrease the magnification if necessary to stop any sensor activity. This will set the optimum highest sensitivity to pick up movement safely. If the setting falls below 6 then contact the supplier as the installation may be an issue. This would apply if airflow or oscillating air mattresses are being used, where additional measures may be needed.
3. On the first few nights leave Shallow Max turned off but set Shallow Min to 0, Shallow Delay=60 and use the Mmd readings to decide on the most suitable settings.
4. Once the user is settled in bed, or sleeping, press the Mm button to reset and start the recorder. The readings on the Guardian's screen will show as M15m15d5. (See pic. below)
5. In the morning, and before the user is awake, note the new readings. Repeat for a number of nights to get a spread of readings and use these to set the alarm values with a margin to prevent false alarms. It is best to begin with Shallow Min=0 with Delay set by the d value as required and prove reliable operation before increasing the Min towards the m value (which can increase false alarms).
6. If the Guardian is an older unit unit that does not have the 'd' reading feature (v6.3), then a reliable value can be found by decreasing the Delay from 60 step by step until an acceptable value is reached without excessive false alarms. Experience suggests most people experience very low breathing levels with gaps in the early morning, which can trigger alarms without any apparent cause.
7. Validate operation during observed seizures and adjust the magnification and detection limits as required. Please call Alert-iT if you have any questions.

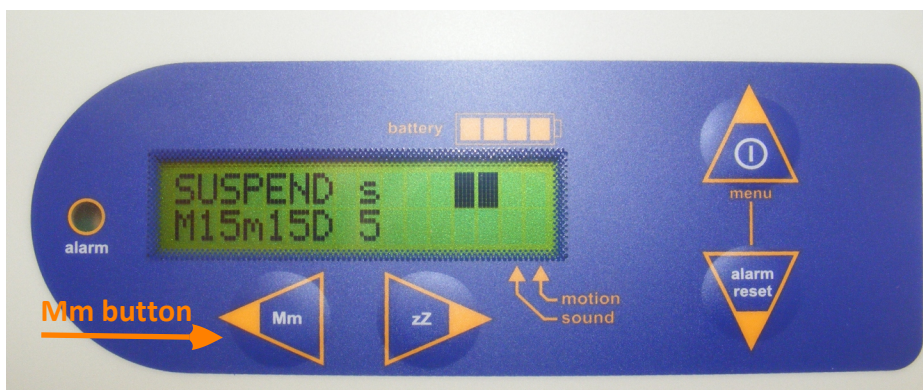

# Shallow Movement Adjustment Procedure

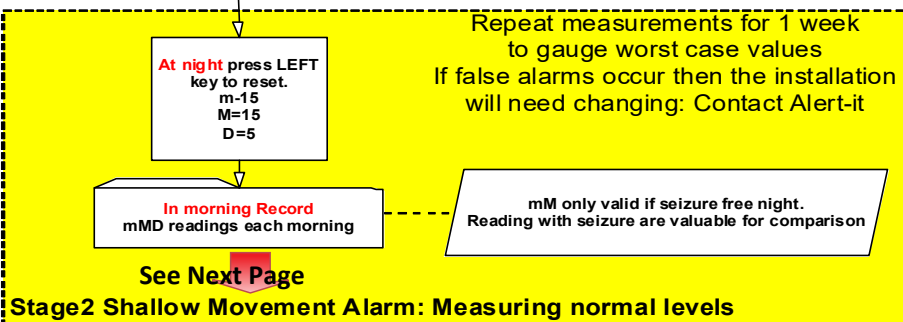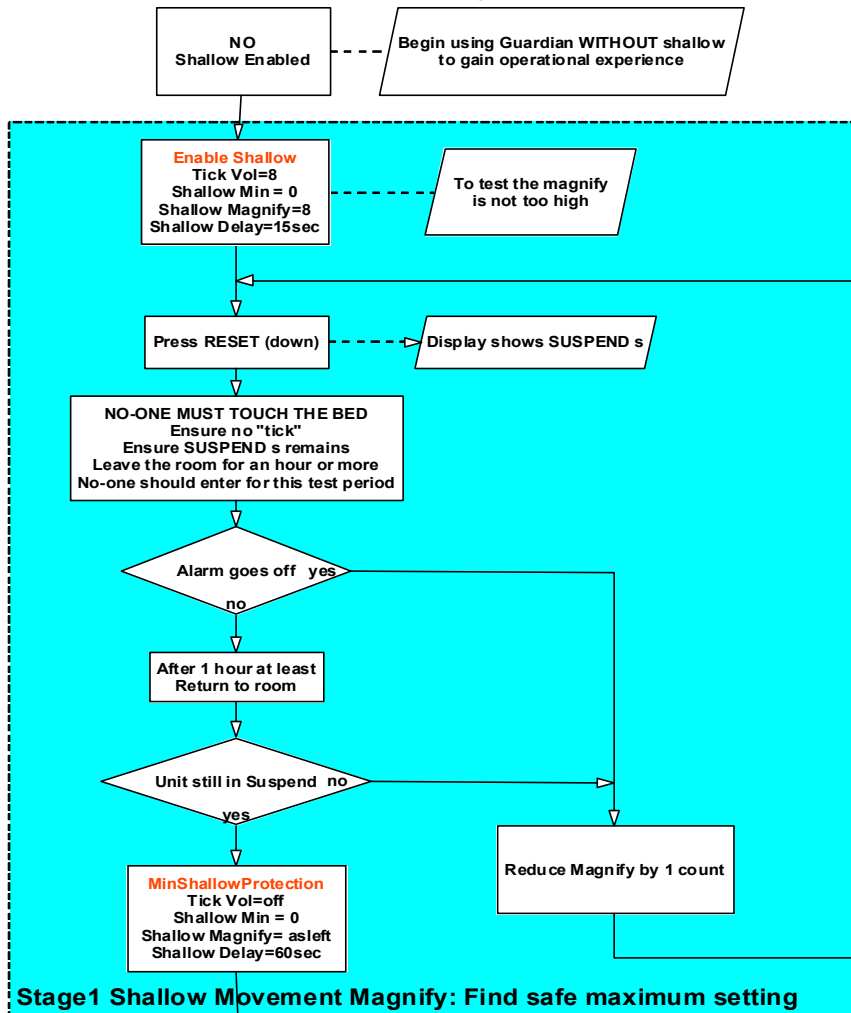



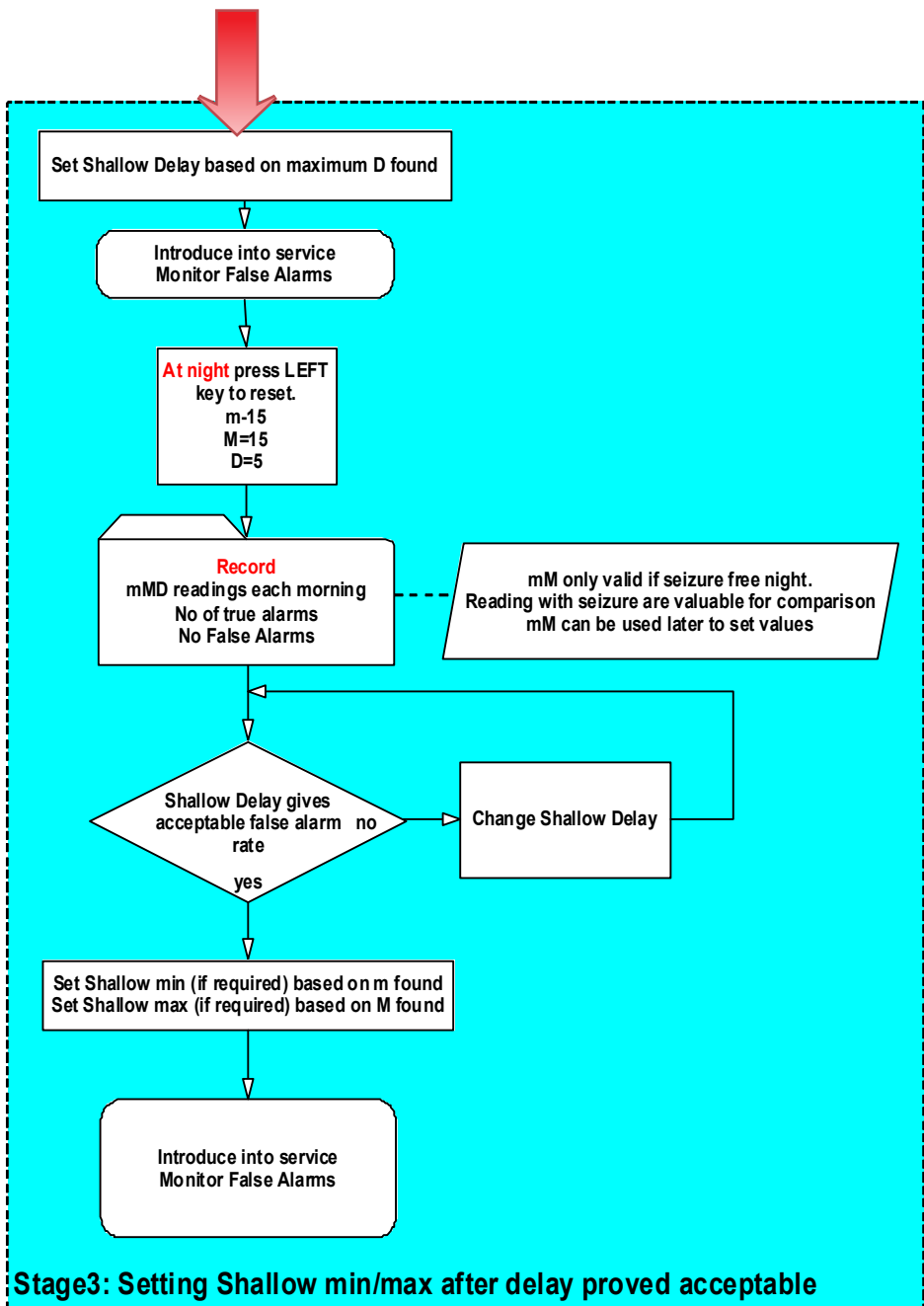

## Bed Vacation

You will require an optional bed occupancy sensor to use this function. Please enquire about our P143 range.



Off as default

When Bed Vacation is enabled, the Guardian is able to detect a bed being vacated for longer than 5 seconds. (This 5 second period is to allow for turning in bed, without causing a false alarm). Bed vacation delay setting allows the monitor to prevent Shallow Movement & Sound alarms while a person is out of bed for a set period of time (up to 24 hr).

Delay time predetermined increments



'Vacnt' displayed when bed unoccupied



### Warning

It is important to ensure that the use of this setting is based upon an individual assessment to ascertain that the user does not move off the sensing area while in bed, as this could prevent the shallow movement detection from operating. For further advice, please contact Alert-iT support.

'InBed' displayed when bed occupied




Alarm activated after set time



## Sound Settings

### Sound Delay


Where seizures can be detected by sound i.e. a clicking of palate, teeth chattering, lip smacking or similar, the Guardian has an internal microphone which can be enabled. N.B. an external microphone can be added as an option (P158B). Settings from 5 seconds up to 60 seconds can be selected, indicating the amount of continuous sound needed to trigger an alarm.

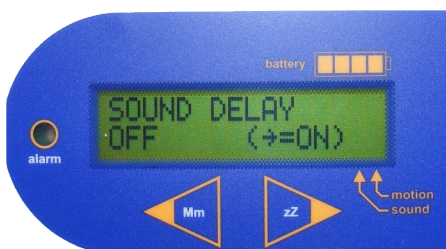

### Sound Magnify


To enable this setting Sound Delay must be ON. To pick up lower volume sounds, increase Sound Magnify up to a maximum of 8. The display will show ) symbol when sound is detected (see figure 6).

Figure 6

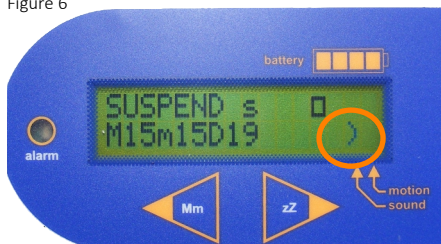

### Sound Rate

To enable this setting Sound Delay must be ON. Sound rate picks up the rhythm of sound. E.g. For default setting 2; one sound in two seconds or faster which will trigger an alarm within the Sound Delay settings timeframe. When sounds are made less frequently increase the number up to 4.

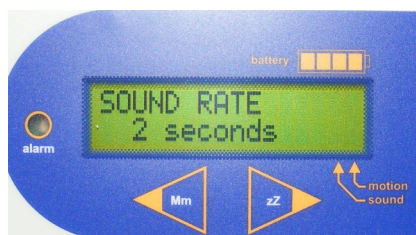

## Moisture

When moisture is enabled, it is used to detect bed wetting, salivation or vomiting. Detection is via optional sensing sheets, which need to be ordered separately to suit your individual's requirements. Please enquire about our P142 range.

Moisture settings from 1 (moist) to 5 (wet) can be set to reflect the requirements of the user.



## Other Settings

### ***Snooze Function Zz***

Whilst in snooze mode ALL alarms will be deactivated for the set period of time. This is useful for personal care or if the user is not settled and false alarms are being activated prior to sleep. To activate the Snooze feature, press the Zz arrow and select the desired time duration. Snooze can be cancelled at any time by pressing the Mm arrow once again - the Guardian monitor is now actively working. Please be aware that whilst in snooze mode the monitor will not pick up any alarms until the set time has elapsed, therefore we recommend extra vigilance during this time.

### ***Reload Default Settings***

Settings can be saved for the individual user. Please refer to Supervisor Handbook for instructions - this is available by visiting the 'Support' page of our website and selecting manual UH1075.

Warning: If "Reload Default Settings" is pressed, this will reload the Factory Default Settings OR the settings saved as per the Supervisor Handbook. Therefore the individual settings, if not saved, will be lost.

### ***Auto-test of sensors***

If a sensor detects no normal activity signal for 24 hours, a fault alarm is raised to prompt a full test by the carer to ensure the sensor has not failed. The Guardian's display screen will show 'Sensor Fault' and can be cleared by activating any sensor - e.g, by tapping the mattress or sitting on the bed.

Alarm Settings						
Function/Menu	Left	Range	Comment	Default	Monitor Screen	Client
ID Number/Site No			Appears on screen at power up with software version			
On/menu key		menu	Press to scroll menu. Press any key also resets alarms		Active or Suspend	
POWER	off		Press to turn off (if enabled)		Blank	
Tick Volume	off	1 8	Volume of click on each shallow movement	OFF		
Shallow Minimum	off	0 10	MPM below which is alarm	OFF	SHALLOW MIN	
Shallow Delay <sup>1</sup>		10 60	Minimum time for Shallow movement alarm to be detected	30	SHALLOW MOVEMENT	
Shallow Maximum	off	20 30	MPM above which is alarm	OFF	SHALLOW MAX	
Shallow Magnify <sup>1</sup>		1 8	Set for optimum movement detection using tick or * symbol	6		
Spasm Delay	off	5 60	Time for spasm to set alarm	15	SPASM MOVEMENT	
Spasm Magnify <sup>1</sup>		1 8	Set for optimum detection of spasm using # symbol	4		
Spasm Rate <sup>1</sup>		1 4	Seconds between movements: a spasm is assumed if faster	2		
Sound Delay	off	5 60	Time for sound level to set alarm	OFF	TRANSIENT SOUND	
Sound Magnify <sup>1</sup>		1 8	Sensitivity of microphone	4		
Sounds Rate <sup>1</sup>		1 4	Seconds between sound bursts: a spasm is assumed if faster	2		
Moisture	off	1 5	Fault alarm if sensor open circuit. Covers moist to wet	OFF	MOISTURE	
Bed Occupancy	off	5 s 24h	Time allowed for vacancy before alarm. Shallow alarm inhibited.	OFF	BED VACATION	
Reload Default			Snapshot saved in "hidden menu"			
Sensor Fault			No activity recorded for 24hrs. Activate any sensor to clear		SENSOR FAULT	

\* 1 Function hidden if alarm off

# Maintenance

## ***Cleaning***

We recommend that you to clean both the Guardian and sensor units regularly by wiping with cotton wool pads moistened (compressed until dripping stops) with a mild detergent (0.5% washing-up liquid) solution, or by using an alcohol or baby wipe. Avoid getting any liquid into sockets or internal areas.

## Compliance

- The system complies with 93/42/EEC as a Class 1 Medical Device for use in a Home Healthcare environment
- The system complies with EN60601 for Class 2 Electrical Safety and does not need a protective earth and Group 1 Class B for EMC in a Home Healthcare environment

## Bibliography

Full handbook: [www.alert-it.co.uk/support](http://www.alert-it.co.uk/support)

## ***Appendix A - Remote Reset Mode***

**Important Note - Remote Reset Mode - Please contact supplier for details of how to activate this function.**

All monitors in Remote Reset Mode will cease transmitting alarms after sounding for sixty minutes continuously. After this point, selecting 'Reset All' on the pager will permanently cancel this alarm.





## Important Safety Information

1. Ensure that any sensor cable is routed and secured to avoid the risk of entanglement or strangulation.
2. Ensure the power cable is routed and secured to avoid the risk of entanglement or strangulation.
3. Regularly check the power supplies for damage and potential shock risks.
4. Ensure, by testing, that the alarm triggers Nurse Call or Third Party Telecare Systems.
5. Regularly test sensors as defined herein.
6. Use only the power supply and batteries recommended.
7. Operate power supply away from direct heat and uncovered.
8. As with all medical electronic equipment there is potential for the equipment to interfere with or be effected by interference from other electrical or electronic devices. For this reason avoid placing the monitor, sensor or connecting cable in close proximity to sensitive electronic devices or devices which produce strong electromagnetic fields such as radio transmitters, mobile phones or power cables.
9. Only use the monitor with accessories approved for use with this product and only in accordance with instructions.
10. If the equipment is modified in any way, appropriate inspection and testing must be conducted to ensure continued safe use of the equipment.
11. The carer must conduct a risk assessment to determine if the level of reliability offered by the monitor is sufficient or if additional monitoring is needed. Contact the manufacturer for assistance with Risk Evaluation Tools.
12. Additional levels of mechanical protection may be needed for some patient disorders. Contact the manufacturers for advice.
13. Some accessories are fitted with small screws and have plastic bags. Ensure these do not come into the possession of vulnerable patients who might choke on them.
14. Any sensor over the mattress (Bed Vacation or Incontinence) has the potential to cause pressure sores . The carer must assess this risk and monitor the use of these products if used.
15. Any sensor over the mattress could pose a fire hazard if in contact with an ignition source.



# Routine Testing Sheet

Serial Number: .....

Test	Signature	Date
1		
2		
3		
4		
5		
6		
7		
8		
9		
10		
11		
12		
13		
14		
15		
16		
17		
18		
19		
20		

*The Alert-it system has been designed with due regard to reliability and integrity. While it offers a highly vigilant monitoring method, it is always possible that a distress condition can go undetected for a variety of reasons (including malfunction) and in life threatening situations it is advisable to use the Alert-it system in conjunction with additional monitoring techniques (e.g. video). Neither the manufacturer nor its agent can accept legal responsibility to provide a system that is infallible. The carer is responsible for assessing the risks of using this equipment and any settings pertaining to it.*



None of the components, including batteries should be disposed of as Domestic Waste. For information on disposal contact ITs Designs Ltd.





### Address

Alert-iT Care Alarms

Fernie House (Unit 3)

Coalville Business Park

Jackson Street

Leicestershire,

LE67 3NR

### Call

01530 239 900

### Visit

[www.alert-it.co.uk](http://www.alert-it.co.uk)

### Social

[www.facebook.com/AlertitCareAlarms/](https://www.facebook.com/AlertitCareAlarms/)

or join our supportive

*Epilepsy Forum*

