

Introducing the P162 Motion Sensor



The battery-operated motion sensor is a perfect solution that can cover the whole of the room, detecting someone leaving their bed before they become weight bearing.

The sensor is discreet with no audible alarm, the device is simply placed on the floor against the wall which avoids trip hazards that other sensor types may cause. The motion sensor is operated by a simple remote control so it can be turned off by the carer when entering the room so not to disturb the user.

P162B (Radio)

The Alert-iT radio version of motion detector is completely wire free and is simple to use and install. It alerts the advanced pager which displays the location of the detector, ensuring a quick response from the carer.

P162AB (Wired)

The Alert-iT wired motion detector enables the carer to create a safe and secure environment. Once the motion detector has been triggered, it will alarm through your nurse call system, which would require a nurse call lead. To identify correct lead please see over leaf.

So simple to use...



STEP 1

The user is lying down and the nurse activates using the remote fob.



STEP 2

User sits up and alarm is raised instantly.



STEP 3

The Carer is notified on the Alert-iT pager or the existing Nurse Call system.

Can be positioned above the bed which detects someone sitting up in readiness to leave their bed.

Another popular use is mounted above a door way "looking" down. In this case the door can be left open and the alarm is raised if the occupant crosses the threshold.

Introducing the

P162

Motion Sensor



P162 Facts

Size:	160x70x40mm
Weight:	220g
Material:	ABS
Battery type:	4xAA
Life of Battery:	Over 1 year on standby. 1 month in constant use.
Electrical Safety:	Class II

Nurse call leads

Aid-Call ATX3000plus, ATX4000 & ATX5000 - P145SC
BLIC (STANLEY) – P145SL
GEMINI – P145SL
INTERCALL – P145G
MEDICARE 6000 (6 PIN) – P145MB
MEDICARE (4 PIN) – P145MA
QUANTEC MK1 – P145SD
SAS NET285 SYSTEM 2 – P145SL
SAS SYSTEM 2 – P145SN (Stereo Connection)
SAS TYPE 1 – P145SD

If your lead isn't showing please visit:

<http://alert-it.co.uk/product/p162ab-wired-pir-motion-detector/>



www.alert-it.co.uk
sales@alert-it.co.uk
01530 239 900



UL1440_June2017v1